



In this document, indicates conditions or practices that could lead to illness, injury, or death in ("WARNING").



Consult Instructions for Use.



Used by trained personnel only.



Do not reuse, single-use device.



Do not use if packaging is damaged.



Not made with natural rubber latex.



Protect from moisture.



Store at room temperature.



Not subject to a sterilization process.



Store out of direct sunlight.



Product Identifier



Lot Identifier



Date of Manufacture



Type B applied part complying with IEC 60601-1.



Fragile



Ingress Protection: Protected from limited dust ingress. Not protected from liquids.

WARRANTY

Please visit www.ceekwomenshealth.com for warranty information. Ceek Women's Health should be notified immediately of any defective units.

DESCRIPTION

The Nella VuLight is a single-use LED illuminator for use with a reusable or single-use vaginal speculum.

INTENDED USE

The Nella VuLight is intended to provide illumination of the vaginal cavity and cervix during gynecologic examinations or procedures.

DIRECTIONS FOR USE

- Remove activation ring just prior to use of VuLight and dispose of ring immediately.
- Apply Nella VuLight to top bill of speculum, ensuring that LED is situated between top and bottom bill.
- Upon completion of exam or procedure, dispose of VuLight in accordance with local regulations.



WARNINGS

WARNING: Inspect packaging and VuLight prior to use. Do not use if damaged.

WARNING: For use only with vaginal speculum.

WARNING: Do not reuse, single-use device.

WARNING: VuLight is not compatible with autoclave sterilization and has not been tested with other forms of sterilization.

WARNING: VuLight has not been tested with laser or electrosurgical equipment.

WARNING: Treat used light as biohazardous infectious material. Dispose of used light in suitable disposal unit or in accordance with local regulations.

WARNING: No modification of this equipment is allowed.

WARNING: California Perchlorate Contamination Prevention Act: the batteries in the VuLight may contain perchlorate materials - special handling may apply. See www.dtsc.ca.gov/hazardouswaste/perchlorate/ for more information.

MANUFACTURED

FOR:

Ceek Women's Health
1455 NW Irving Street,
Suite 200

Portland, OR 97209

www.ceekwomenshealth.com

